# vario DIGITAL FHD Advanced

1659-16, 1659-161, 1659-162, 1659-163

Operating instructions

EN

Mode d'emploi

FR

Manual de instrucciones

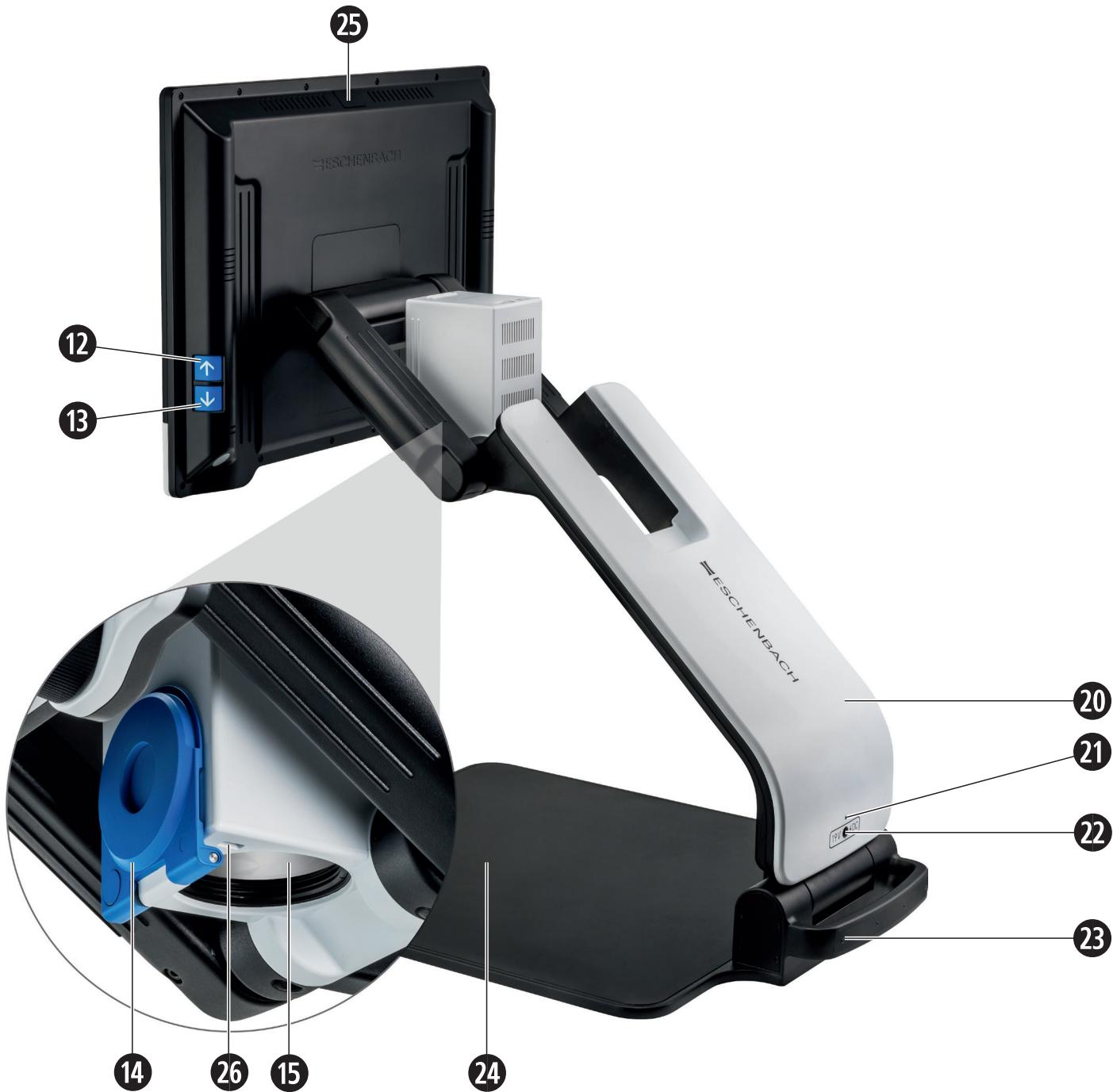
ES

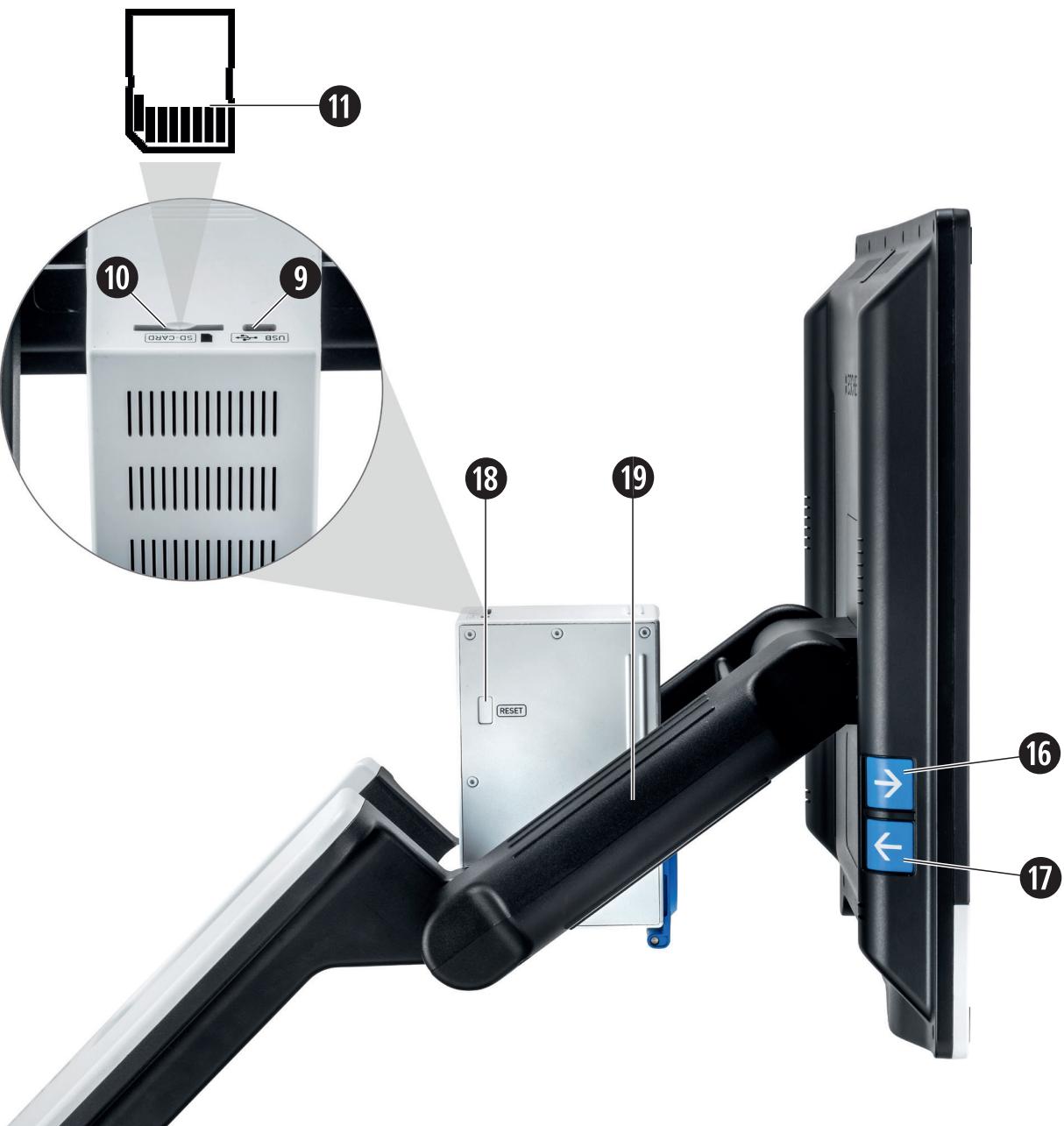
CE

ESCHENBACH

English .....	5
Français .....	27
Español.....	49

A2

Dear Customer,  
Congratulations on the purchase of your Eschenbach vario DIGITAL FHD Advanced.  
As market leader in magnifying glasses and reading aids we at Eschenbach can look back  
on many years of experience, which is reflected in the quality, handling and design of our  
products.

EN

## Firmware version

These instructions are based on firmware version "VD16Adv\_fw\_1.1" or higher.

## Intended use

This electronic reading device is exclusively intended for magnification and reading or viewing of texts and images. To guarantee correct functioning, the screen reader must be set up as described.

The product is operated in a domestic environment (also includes nursing homes, workplaces, schools and other educational facilities).

## Before first use

Please read these operating instructions thoroughly before using the device for the first time. This is the only way to get to know all the features available for the device.

## Package contents

- ▶ vario DIGITAL FHD Advanced
  - ▶ Remote control with 2 × AAA batteries
  - ▶ Power adapter
  - ▶ HDMI cable
  - ▶ USB cable (Type C)
  - ▶ Microfiber cloth
  - ▶ Operating instructions
- 
- ▶ Depending on the version:
    - incl. battery
    - incl. X-Y table

## Safety information

EN

- ▶ **Risk of damage to the eye and risk of injury:** Never look into the light sources on the underside of the device.
- ▶ **Never allow children to handle electric devices without supervision.**
- ▶ **Risk of suffocation and risk of accidental swallowing:** Keep packaging, films and packaging parts away from children.
- ▶ **Risk of pinching:** ensure that your fingers do not get pinched during unfolding and folding of the device, and also during unfolding and folding of the camera on it.
- ▶ **The entire device must stand on a stable surface.**
- ▶ **Take care not to leave any objects on the base when you want to fold the device together. This can result in damage to the display.**
- ▶ **Protect the device against impact or shock. Handle the device with care.**
- ▶ **Never subject the device to humidity or moisture.**
- ▶ **Protect the device from excessive heat and open flames. Never place the device on radiators or in direct sunlight.**
- ▶ **Do not store the device in locations with excessive heat, for example in a parked car.**
- ▶ **The light source contained in the device may only be replaced by the manufacturer, its customer service department or another qualified person.**
- ▶ **Use only the supplied power adapter and country-specific power plug adapter.**
- ▶ **Check the device, the power adapter and its cable for visible damage. If there is any damage, disconnect the power adapter from the power supply immediately. Have the product checked by a qualified technician before you use it again.**
- ▶ **Class 1 laser in acc. w. EN 60825-1:2014.**
- ▶ **Route the power cord so that it does not present a tripping hazard.**
- ▶ **Never open the device housing. Opening the housing voids the warranty.**

- ▶ The device must be turned off and the power plug must be removed before it is folded together.
- ▶ Carry the device only when it is folded together and by using the carrying handle.
- ▶ Only for domestic use.
- ▶ In a medical environment this device may not be operated together with other devices that are used for life support.
- ▶ Ensure that other people, especially children and persons who cannot assess this hazard, are aware of this hazard.
- ▶ The device is a Class I medical device and complies with Directive 93/42 EEC, Annex IX, Regulation 12. When using the device outside the EU member states, the respective country-specific directives must be observed.
- ▶ If you purchased the device including batteries, they may only be removed or exchanged by authorized personnel.
- ▶ CAUTION risk of explosion! Never replace the battery with the wrong type. Never dispose of a battery in a fire. Never crush or cut a battery. Never leave a battery laying around in an environment with extremely high temperatures, as this can result in an explosion or cause flammable liquids or gases to escape. Never subject a battery to extremely low air pressures, as this can result in an explosion or cause flammable liquids or gases to escape.
- ▶ When using the variant with X-Y table:
  - Make sure not to get your fingers caught when moving the table.
  - Only ever carry the device on the intended carrying handle when it is folded together. Fasten both brakes and press the button to lock the X-Y table (see page 24).

# **Functional elements**

(For illustrations see pages A1 and A2.)

EN

1. Display
2. Rotary knob to open contrast functions
3. Photo button for saving images and confirming menu options
4. Button for turning the reading line on and off
5. On/Off button
6. Button for opening menu operation
7. Button for overview display
8. Rotary knob to increase or decrease zoom factor
9. USB port
10. SD card slot
11. SD card (insert card as shown)
12. Button for moving image selection upward
13. Button for moving image selection downward
14. Foldable camera cover (flipped open in illustration)
15. Camera
16. Button for moving image selection to the right
17. Button for moving image selection to the left
18. RESET button for resetting the device
19. Monitor arm for setting screen height and tilt angle
20. Base arm for unfolding the device
21. Charging control LED
22. Connection for power adapter
23. Carrying handle
24. Base plate
25. HDMI input
26. Rangefinder

# Basic operation

## Structure of the device and power supply

Place the device on the base plate ②⁹. Unfold the base arm ⑩ on the device until it engages (see illustration). The camera ⑯ is equipped with a cover ⑮. Open the camera cover and move it to the intended position (see illustration on pages A1 and A2). If the  symbol is shown, the camera cover has not been opened.

Use the joint ⑨ to set the required screen height and tilt angle.

Connect the power adapter to a suitable device plug. Connect its cable to the connection for the power adapter ⑪. Then plug the power adapter into the power socket. The charging control LED ⑫ lights up while the device is being charged.

To fold the device together again move the screen backwards and downwards. Then fold the camera together and close the camera cover (see illustration). Thereafter, fold the base arm downward.

## Turning on/off

Press the On/Off button ⑤. The camera image will appear on the screen ① after a few seconds. Symbols briefly show the current status, for example magnification factor and brightness setting.

To turn the device off, press and hold the On/Off button ⑤ for one second. Current settings are saved.

## Reading with the device

Turn the device on and place the reading material on the base plate ② under the camera ⑯.

In exceptional cases it can make sense to relocate the field of vision to the upper area of the base plate. To do this, swivel the camera out past the locking position. It is also possible to move the camera forwards, for example for carrying out handicrafts.

## Adapting the magnification

To increase the magnification turn the rotary knob ⑧ in clockwise direction when the live image is displayed. To decrease the magnification, turn the rotary knob ⑧ counterclockwise. The magnification factor you have set will be displayed briefly.

EN

## Overview function

If you would like to see an overview of the reading material, then press the  button ⑦. The magnification will change to 1.3x. If you press the  button again, the previously set magnification will be selected again.

## Dynamic Line Scrolling (DLS)

With the aid of the DLS function, you can move the image selection horizontally or vertically within the visible area without having to move the reading material.

A reading mode must first be selected before you can use DLS. This then defines the image selection available. There are three reading modes: Letter, Book und Column. You should select the mode which best matches the width of your reading material. Each of the reading modes has a different basic magnification.

- ▶  Reading Mode 1: Letter (1.6 times – maximum 12 times magnification).  
Use this reading mode for very wide objects (up to 215 mm).
- ▶  Reading Mode 2: Book (2.5 times – maximum 20 times magnification).  
To read a book (up to 130 mm wide) with the aid of DLS, select this mode.
- ▶  Reading Mode 3: Column (5 times – maximum 37 times magnification).  
To read (newspaper/magazine) columns (70 mm) with the aid of DLS, select this mode.

Please proceed as follows to activate DLS:

Hold the  button ③ down for about 3 seconds. Now you can use the ② or ⑧ knobs to choose between three reading modes.

To confirm, press the  button ③. The device then zooms to the preset width. That is the field of vision available for DLS. If you press the  button ③ again the menu will be hidden. Now you can adapt the zoom factor by using the ⑧ knob.

With buttons ⑫, ⑬, ⑯ und ⑰ you can move the displayed section horizontally or vertically without having to move the reading material itself. The image selection can be moved upward with arrow key ⑫, downward with arrow key ⑬, to the right with arrow key ⑯, and to the left with the arrow key ⑰. If you press one of the arrow keys twice in rapid succession, the screen section is moved to the respective end. For example, you can quickly find the beginning of the next line by pressing the ⑰ arrow key twice.

To exit the reading mode, hold down the  button ③ and select the  symbol. Then press the  button ③ again to close the menu. Thereafter, the entire magnification range is available again.

The selected reading mode is reset when the device is turned off.

Alternatively, the DLS can also be operated using the remote control (see "Remote control" chapter on page 12).

## Turning the reading line/row on and off

Press the ④ button to turn the reading line on. Press it again to hide the line.

## Special button functions

- ▶ Press and hold the  button ③ to activate/deactivate DLS (see "Dynamic Line Scrolling (DLS)" chapter on page 10).
- ▶ Press and hold the  button ⑦ to turn the autofocus function off (see also Chapter "Deactivating/activating autofocus" on page 12).
- ▶ Press and hold the reading line button ④ to change the reading material brightness (see Chapter "Adjusting the reading material brightness" on page 18).

## False color display

To display the camera image in false colors for increased contrast, turn rotary knob ② until the required false color rendering is displayed. By default, the false colors are preset

as black/white and white/black. To adapt the false color display, open the "Adjusting false colors" menu (see Page 18).

## Saving the live image as a file

EN

Press the  button ③ briefly to save the current camera image as a file on the inserted SD card and to show it as a still image on the screen. You can also change the magnification factor for this image as already described. You can use the arrow keys ⑫ + ⑬ to move the magnified image section vertically, and arrow keys ⑯ + ⑰ to move it horizontally. Press the Photo button again to return to the live image display.

## Deactivating/activating autofocus

Press and hold the  button ⑦ to deactivate the autofocus function. This can be helpful, for example while writing under the device, so that the focus remains on the reading material. Press the  button again to reactivate autofocus. Autofocus is automatically reactivated when the device is turned off.

## Mirror function

If the camera is pointed toward the user and the screen is lowered slightly, it is possible for the user to look at himself. This can be useful, for example to apply make-up or for facial care.

## Remote control

The remote control can be used while you are holding it in your hand and also when it is lying on the table. The anti-slip strips ensure that the remote control grips the table well. The remote control only works if there is a visual connection between yourself and the IR sensor below the display.

## Setting-up for use

One battery terminal is insulated against the contact area with a plastic strip to avoid self-discharge. To remove it, open the battery compartment cover on the underside of the remote control and pull the strip out.

## Adapting the zoom

To zoom in, press **A**. To zoom out, press **B**. The currently selected zoom factor is shown briefly in the display.

## Dynamic Line Scrolling

To use Dynamic Line Scrolling, press and hold the OK button **F** briefly. Then you can select the desired reading mode using the arrow keys < and > **E**. Once the desired zoom factor has been set, you can use the arrows to move the displayed section horizontally or vertically without having to move the reading material itself. If you press the arrow buttons at the end/beginning of the image selection for 2 seconds, the image selection jumps to the other end/beginning. This makes it much easier to read text.

## Overview function

If you would like to get an overview of the reading material, press  button **C**. The magnification changes to 1.3 x. By pressing the  button **C** again, the previously set magnification is selected again.

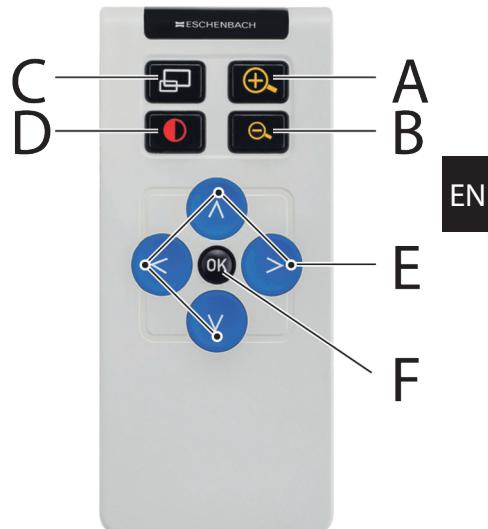
## False color display

To change false colors, press the **D** button until the desired false color is displayed.

## Channels


A switch for switching over the channels is located in the battery housing. However, this is only necessary if two devices are being used next to each other. You can set the relevant channel in the menu under "Remote control" (see page 19).

The channel selected on the remote control and in the menu must be identical. Only then can the remote control be used. Channel 1 is used by default.

# Opening the main menu

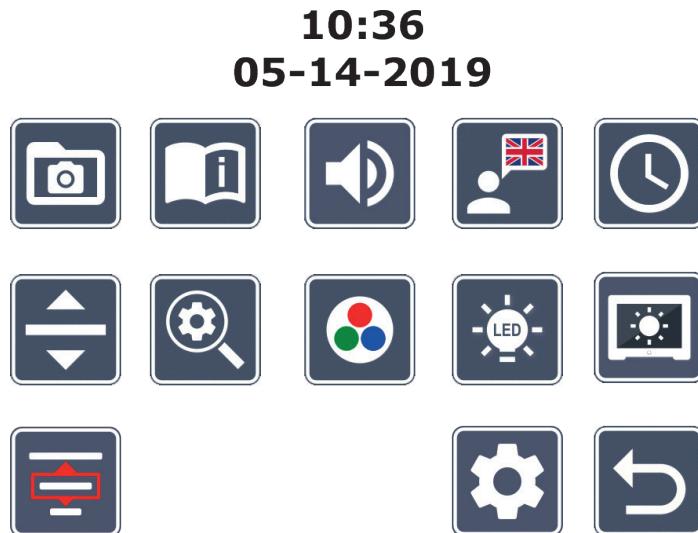Press the MENU button ⑥ to open the menu:

You can move to the individual symbols using the rotary knobs ② + ⑧ and the arrow keys ⑫, ⑬, ⑯, ⑰. The symbol for the selected function becomes larger and is displayed in colour. Press the button ③ to activate each selected function.

Individual menu commands are also played back acoustically. Playback can also be deactivated by changing the volume. To do this, press the symbol several times until the symbol is displayed.

If you press the button ⑦, the currently selected menu item is audibly played back again.

You can exit the setting menu at any time by simply pressing the MENU button ⑥.

## Managing image files on the SD card

Highlight this symbol and then press the button to open image files from the SD card.  
The following menu appears:

EN

The currently selected image file will be displayed centrally above it. The previous image is shown to the left of it, the next one to the right.

Use the two rotary knobs ② + ⑧ to select the individual symbols with the following functions:

- select the previous image
- select the next image
- select or deselect the image currently being displayed in the center as a favorite.

**Tip:** the image selected as a favorite is highlighted with a yellow frame and will be displayed first when the photo album is opened. Only *one* image can be selected as a favorite at any one time.

- delete the selected image from the SD card
- back one level

Press the button ⑦ to display the image file in the middle full screen. You can use the rotary knob ⑧ to set the required zoom factor. You can use the left-hand arrow key ⑯ + ⑰ to move the zoomed-in image section horizontally and the right-hand arrow key ⑫ + ⑬ to move it vertically.

Press the button ⑦ again to quit full-screen display.

## Deleting image files

To delete the selected image file from the SD card, first select the delete symbol  and then confirm with the  button ③. The confirmation symbol  is now shown on the bottom right of the screen, and the cancel symbol  is shown on the left. You can use the rotary knobs ② + ⑧ to select  or  button to execute the selected function.

## Opening the quick guide

Select the national flag that represents the language you desire. Press the  button to change the language. You can use  /  to scroll the page preview backward and forward. Operation is analog to viewing images from the SD card. Press the  button ⑦ to quit full-screen mode.

## Adjusting volume

The current volume setting is displayed. Select this symbol and then press the  button ③ to select the following settings:

- very low volume

- low volume

- medium volume

- high volume

- mute

## Selecting the language for voice output

Select this symbol and then press the  button ③ to select the language for voice output. The symbol for the current voice output setting is displayed. Alternatively you can select  or  for confirmation using a signal tone.

The  symbol is displayed while the language file is being loaded.

## Setting the date and time

The time will only be displayed in the menu once it has been set. Select this symbol and then press the button ③ to set the time. Select the symbol so that you can increase the number of the parameter displayed in yellow using the button ③. Select the symbol so that you can decrease the number of the parameter displayed in white using the button ③. Select the symbol and then press the button ③ to select the next parameter to be set.

Select "24H" and press the button ③ to change from a 24-hour display to the 12-hour display normally used in English-speaking areas or vice versa.

The date format normally used in the EU is set as standard. Select EU and use the / symbols to change the date format if required.

## Adjusting the reading line/row

Select this symbol and then press the button ③ to set the details for the reading line or row using the following symbols:

- move the reading line position upward or magnify the reading row

- move the reading line position downward or minimize the reading row

- turn the reading line/row on/off

- switch to "red reading line" or "reading row"

- switch between horizontal/vertical reading line/row

Alternatively you can use the ⑫ and ⑬ arrow keys to change the position of the "reading line" or "reading row".

## Adjusting maximum magnification

Select the symbol for the maximum magnification factor that should be available.

The selected zoom factor is ignored in DLS mode.

## Adjusting false colors

Select this symbol and then press the button ③ to select the false color combination that should be available. Black/white plus inverting are activated by default.

EN

Non-activated combinations are shown crossed out.

Select the required combinations and then change their status using the button ③.

## Adjusting the reading material brightness

Select this symbol and then press the button ③ to set the brightness for the reading material using the following symbols:

- no LED lighting

- the LEDs light up with half power

- the LEDs light up with full power

- turn the LED spots on the screen on or off

Confirm the required brightness using the button ③.

## Adjusting the screen display brightness

Select this symbol and then press the button ③ to adjust the display brightness using the following symbols:

- display brightness = 25 %

- display brightness = 50 %

- display brightness = 75 %

- display brightness = 100 %

## Selecting the reading mode

Select this icon to set the reading modes described in the “Dynamic Line Scrolling (DLS)” chapter.

EN

## Opening the second menu level

Select this icon and press the button ③ to enter the second menu level.


## Quit the menu

Select this icon and press the button ③ to return to the live image display.

## Second menu level

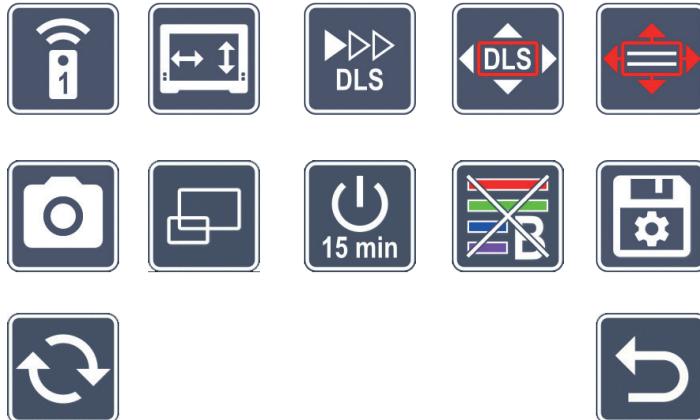The second menu level provides the following functions:

## Remote control

Controlling two devices remotely in the same room may cause them to interfere with each other. This menu allows the user to set different communication channels for the remote control to prevent interference.

The channel selected on the remote control and in the menu must be identical. Only then can the remote control be used. Channel 1 is used by default.

EN

### Swapping navigation keys

Select this icon and press the button ③ to switch the functionality of the lateral key pairs for moving the image selection. Horizontal movement is then carried out using the right-hand button pair, vertical movement using the left-hand button pair.

### Setting the reading speed

Select this icon and press the button ③ to set the scroll speed at which the camera image moves across the display when using the image scrolling buttons:

- low scroll speed

- medium scroll speed

- high scroll speed

### Enabling/disabling DLS

Select this icon and press the button ③ to enable the Dynamic Line Scrolling (DLS) function. If the function is disabled, the symbol is crossed out. If you would like to enable the function, press the button again.

### Enabling/disabling vertical DLS

Select this icon and press the button ③ to disable vertical Dynamic Line Scrolling (DLS). If this icon is shown, only horizontal DLS is available.

### Enabling/disabling the photo button

Select this icon to enable or disable the photo function of the button ③. If the photo button is disabled, the symbol is crossed out. If you would like to enable the button, press the button again.

## Enabling/disabling the overview display

Select this icon to disable the overview display function. If you would like to enable the overview display again, press the **O** button ③ again.

EN

## Changing the timeout settings

As standard, the device switches off automatically after approx. 15 minutes if it is not used. Select this icon and press the **O** button ③ to extend this period to approximately 8 hours.

## Enabling/disabling the blue filter

The blue filter is disabled as standard. Select this icon and press the **O** button ③ if you want to enable it.

If the blue light filter is enabled, the amount of blue light is reduced. The display becomes more yellow and the colors appear softer.

## Saving settings, loading and restoring factory settings

Select this icon and press the **O** button ③ to manage menu settings such as language, volume, activated false colors, LED settings, etc.:

- The first function allows you to save your current settings.
- This function loads the settings you saved previously. The requirement for this is that settings were saved previously.

**Tip:** Once you have set your device up for the first time you can restore your personal configuration using these two functions.

- This function allows you to reset the device to the default settings if required. All settings will then be as set at the default settings.

## Updating the firmware

Select this symbol and then press the button ③ to display the current version of the firmware on the device and to update it if necessary. If you have registered your vario DIGITAL FHD Advanced, you will be notified about any available firmware updates.

To update the firmware please proceed as follows:

1. Connect the vario DIGITAL FHD Advanced to a PC using the provided USB cable. Ensure that the SD card is inserted in the card slot ⑩.
2. Turn the vario DIGITAL FHD Advanced on. The contents of the SD card will now be displayed on the PC as the "ESCHENBACH" drive.
3. Now open the web address that was provided to you in the email about the firmware update using the connected PC.
4. Download the firmware update and save it directly on the drive - not in a drive subfolder.
5. Disconnect the vario DIGITAL FHD Advanced from the PC using the "Unplug and eject hardware" function, whose symbol you can find on your operating system taskbar.
6. Open the settings menu, select the symbol and press the button.
7. Select the symbol and press the button. To cancel the update and return to the previous page, select the symbol and press the button.

The following icon will be displayed on the screen during the firmware update:

The vario DIGITAL FHD Advanced will turn off automatically once the update is successful. **Never turn the device off and never remove the SD card from the device during the firmware update.** Otherwise the vario DIGITAL FHD Advanced could be irreparably damaged. This type of damage is not covered by the warranty!

You can use the symbol and confirm it with the button ③ to quit this menu level and return to the main menu.

# Connections

## USB port

You can use the supplied USB cable to connect the device to the USB port on a computer using the device USB port ⑨. The SD card inserted in the device will then be displayed on the computer as the "ESCHENBACH" drive for file operations.

EN

## HDMI

You can use the enclosed HDMI cable to connect the device via its HDMI input ⑯ to the HDMI output of other devices. If the device detects the HDMI signal at the input, it automatically switches to the display of this signal. To switch back to the camera display, press and hold the MENU button ⑥ for 3 seconds. To switch back to the HDMI signal, press and hold the MENU button ⑥ for 3 seconds again. If the connection to the HDMI device is interrupted, the device automatically switches to the camera display.

This function depends on the output device. The camcorder or the camera must be fitted with an HDMI output and HDMI Live.

If you do not need sound, you can set this in the menu. Open the menu and set the volume () to muted ().

## Reset

If the device no longer functions as described in these instructions, it may be due to a malfunction, which you can eliminate yourself by resetting the device. Then press the RESET button ⑯.

# Additional equipment variants

## Battery

If you own the vario DIGITAL FHD Advanced with a battery, please read this paragraph carefully:

- The battery may only be removed or exchanged by authorized personnel.
- It takes 3.5 hours to completely charge the battery, and this allows an operational duration of 4.5 hours.

## X-Y table

If you own the vario DIGITAL FHD Advanced with X-Y table, please read this paragraph carefully:

27. Horizontal brake (unbraked)
28. Vertical brake (unbraked)

29. Button for locking the X-Y table
30. Button for releasing the lock

Before you can use the X-Y function, press the button to release the lock 30. Then, move the brakes 27 and 28 to the shown position. You can now move the X-Y table horizontally and vertically.

If you only move brake 27 right, the X-Y table can only be moved vertically. If you only move brake 28 up, the X-Y table can only be moved horizontally. For transport, only ever carry the device on the intended carrying handle when it is folded together. Fasten both brakes 27 and 28 and press the button to lock the X-Y table 29.

EN

## Technical specifications

Display size:	39.6 cm (15.6")   16:9   Full HD
Camera:	optical/digital zoom camera with AF
Magnification range:	1.3 × - 45 ×
Display modes:	real color, contrasting up to 14 false color combinations possible
Card formats supported:	SD, SDHC (max. 32 GB)
Charger power supply:	Primary: 100 V – 240 V ~   50 – 60 Hz Secondary: 19 V ==   2000 mA
USB port:	Type C (only for data transfer)
HDMI input:	HDMI 1.4, depending on output device
Operating temperature range:	+10 °C – +40 °C (50 °F – 104 °F), humidity: < 80 %
Storage temperature range:	0 °C – +60 °C (32 °F – 140 °F), humidity: < 80 %
Dimensions:	378 × 460 × 132 mm (14.9" x 18.1" x 1.6") (W × H × D)
Dimensions with X-Y table:	435 x 500 x 160 (W x H x D)
Weight:	~ 6.3 kg ~ 6.6 kg (incl. battery) ~ 8.7 kg (incl. X-Y table) ~ 9.0 kg (incl. X-Y table and battery)
Lighting:	LED
Power consumption:	~ 16 W
Images per second.:	60 fps
Contrast ratio:	1000 : 1

These instructions are also available on the Internet at <http://www.eschenbach.com>

USB® is a registered trademark of USB Implementers Forum, Inc.

## Care instructions

EN

- ▶ Turn off the device and disconnect it from the power source before you clean it.
- ▶ Clean the device using a soft, dry cloth only.
- ▶ Never use soap solutions containing plasticizers or alcoholic or organic solvents to clean your device, and never use abrasive detergents.
- ▶ Never clean the device in an ultrasonic bath or under running water.
- ▶ Protect your device against heavy contamination.
- ▶ Condensate may form and cloud the lens when you move from cold rooms to warm rooms. If this happens, wait 40 minutes before starting the device to avoid condensate formation.

## Disposal

- ▶ This product is subject to the provisions of European WEEE Directive covering old electrical and electronic devices in its currently applicable version. Never dispose of your old device in normal domestic waste. Instead dispose of it through an approved disposal facility or your municipal waste facility. Observe the regulations applicable in your country. In case of doubt, contact your waste disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

## Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. No warranty claims can be accepted for damage that arises due to improper handling, including damage from falls or impacts. Warranty claims can only be accepted on presentation of the sale receipt.

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USB® es una marca registrada de USB Implementers Forum, Inc.
Eschenbach Optik of America, Inc.